


# Wall Mount 4

alcohol tester

# Quick User Guide

1

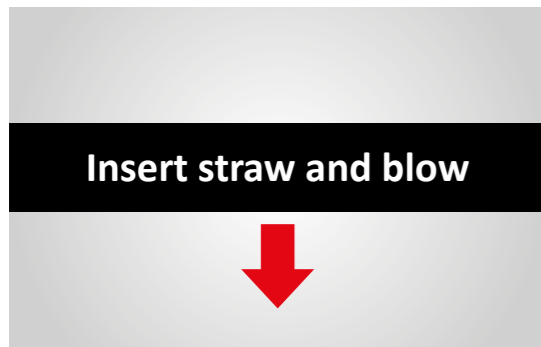


It is recommended the subject **not consume food, drink or smoke 15 minutes prior to taking a test.**



The instrument is ready to receive a breath sample at any time – follow the screen prompts to begin.

2

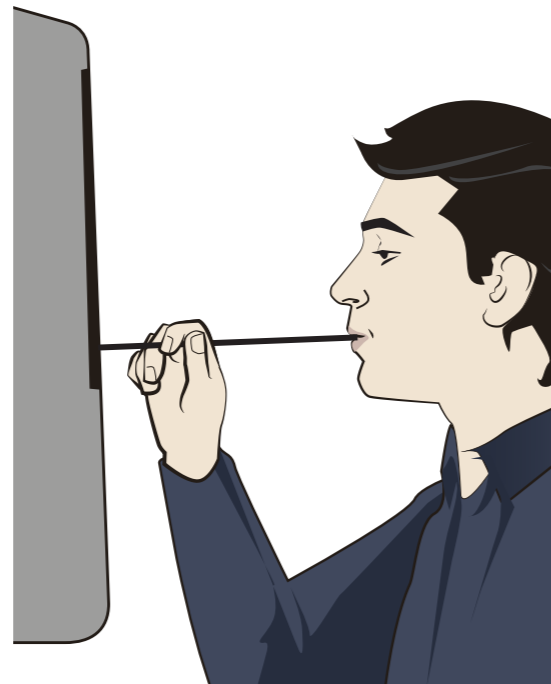

Take a straw from the straw holder.  
Place the straw into the white circular Sample Port located immediately below the screen.

3




Take a deep breath and blow steadily into the straw. Follow the screen prompts until the screen displays “**Stop blowing**”.

You will hear clicks and a long beep – the test is then complete.

The device will analyse and then display a colour coded numerical BAC result.

4

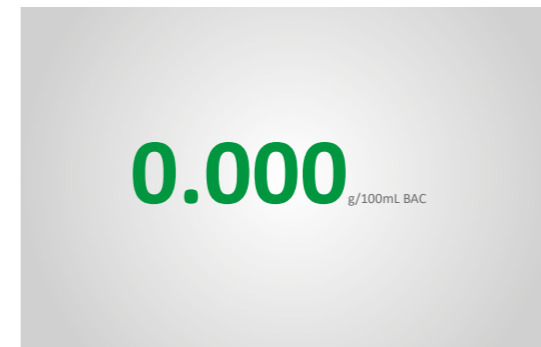


A screen with **green numbers** indicates that **alcohol is not present.**

A screen with **orange numbers** indicates that **alcohol is present but currently under locally accepted BAC levels.**

A screen with **red numbers** indicates that **alcohol is present and currently over locally accepted BAC levels.**

Follow relevant instructions as designated by the local municipality and/or your employment terms.

5

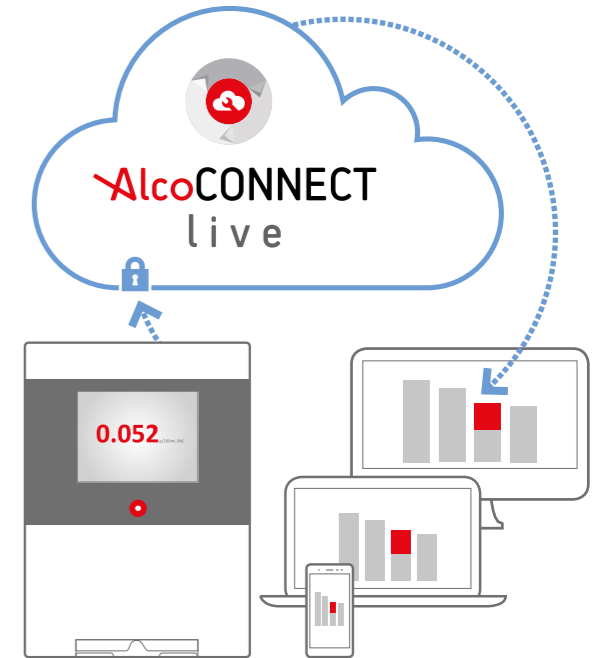

When AlcoCONNECT Live is enabled, Alcolizer Wall Mount 4 delivers real-time analysed test data from across your business.

Alcolizer Wall Mount 4 connectivity options also enable:

- saving of test data
- send an email and/or direct an employee/contractor when an exception test occurs
- action gate access
- integration with your site management systems