

AlcoCONNECT

data management

LIVE DRUG & ALCOHOL DATA MANAGEMENT

- ✓ Drug and Alcohol test data at your fingertips
- ✓ Dashboards with real-time data analysis
- ✓ Monitor compliance in daily testing programs
- ✓ Identify risk areas with drill-down capability
- ✓ Export data for reporting
- ✓ Multi-factor security for maximum data integrity

AlcoCONNECT™ Live Data Management is an essential tool for Safety and Business managers seeking real-time, analysed Alcohol and Drug test results from across their business.

The AlcoCONNECT™ Live Data Management system combines Alcolizer's innovative testing technology with state-of-the-art business solutions.

ACHIEVE YOUR DRUG & ALCOHOL PROGRAM OBJECTIVES

We understand that every work place strives to be drug and alcohol free.

Alcolizer's AlcoCONNECT™ Live Data Management system provides you with real-time analysis on the effectiveness of your drug and alcohol program.

Understanding where and how exception tests occur enables you to target where additional training and support is required. By doing so you can ensure all employees are best able to comply with your site's safety requirements.

You will identify hot spots in your business and target resources so you can maximise the effectiveness of your testing spend.

RELIABLE, FAST, CONFIDENTIAL ACCESS

Drug and alcohol tests undertaken by connected Alcolizer Technology devices are available confidentially on our AlcoCONNECT™ Data Management portal.

Management can view the data for their site, while national managers can view tests from every site.


EASY COMPLIANCE WITH ELECTRONIC DATA – NO MORE PAPERWORK!

AlcoCONNECT™ Data Management records all drug and alcohol test data onto a secure electronic database.

Compliance reports are generated from this information and email alerts are sent to management when an exception test is recorded.

DATA SECURITY IS ASSURED

Alcolizer Technology ensures that your confidential data is available 24/7 and is fully-secure. The system utilises a Secure Sockets Layer, link encryption and Multi-factor authentication via a data centre in Sydney.

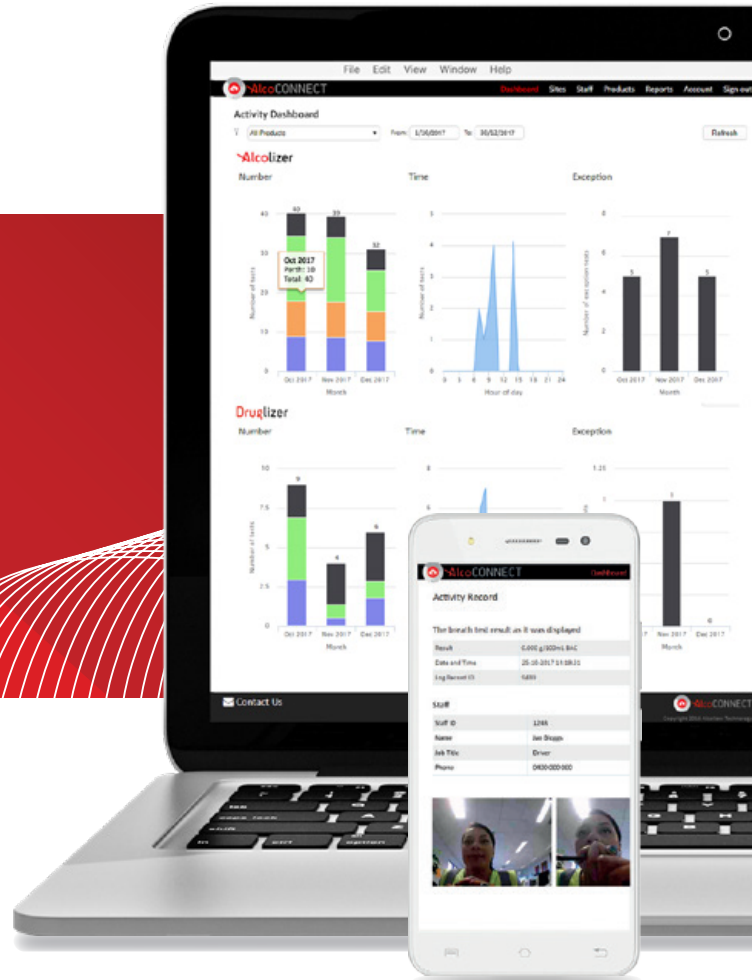

FEATURES

- Remote results monitoring
- Real-time alerts
- Instant access from anywhere in the world
- Dashboard user interface for at-a-glance results accessibility
- Automatic service and technical issue alerts delivered directly to Alcolizer Technology factory trained engineers
- Multiple-location synchronisation
- Secure cloud-based test results storage



Quality
ISO 9001:2015
SAI GLOBAL

AlcoCONNECT

data management



AT-A-GLANCE DASHBOARDS

AlcoCONNECT™ Live dashboards provide an easy at-a-glance view of your test data by number of tests, site location, time of day and result.

Alcolizer and Druglizer test data

Drug and Alcohol test data – logged by various Alcolizer Technology alcohol and drug testing equipment – is displayed separately in three graph data sections, *Alcolizer*, *Druglizer* and *OnSite Testing*. These graphs provide the following data summaries:

- **Number** – the number of readings by month, grouped by Site.
- **Time** – the number of readings by time taken.
- **Exception** – the number of exception readings by month, grouped by Site.

ACTIVITIES LISTS AND RECORDS

Drill down into this data for access to original alcohol, drug screen and lab confirmation results – do so by clicking (or tapping) the *Number* and *Exception* coloured segments to open an *Activities* List.

Click/tap an entry in the list to open an *Activity Record* screen – this is where you can view all details of the test record.

Only users that have been set up with appropriate access will be able to view the data within these records.

Note: camera photo capture is an optional feature only available on an Alcolizer Wall Mount 4 or Centurion.


REPORTS, AUTOMATION AND MANAGEMENT

Managers with appropriate permissions can access and download various reports such as Breathalyser Activity, Staff Activity and OnSite Testing Activity. Emailing of reports can be also automated and, when coupled with an internet connected Alcolizer wall mount device, users can receive an email when an exception is recorded.

Managers can upload and maintain staff lists containing information such as staff positions and identification data.



Configurable Graph and Map Dashboards

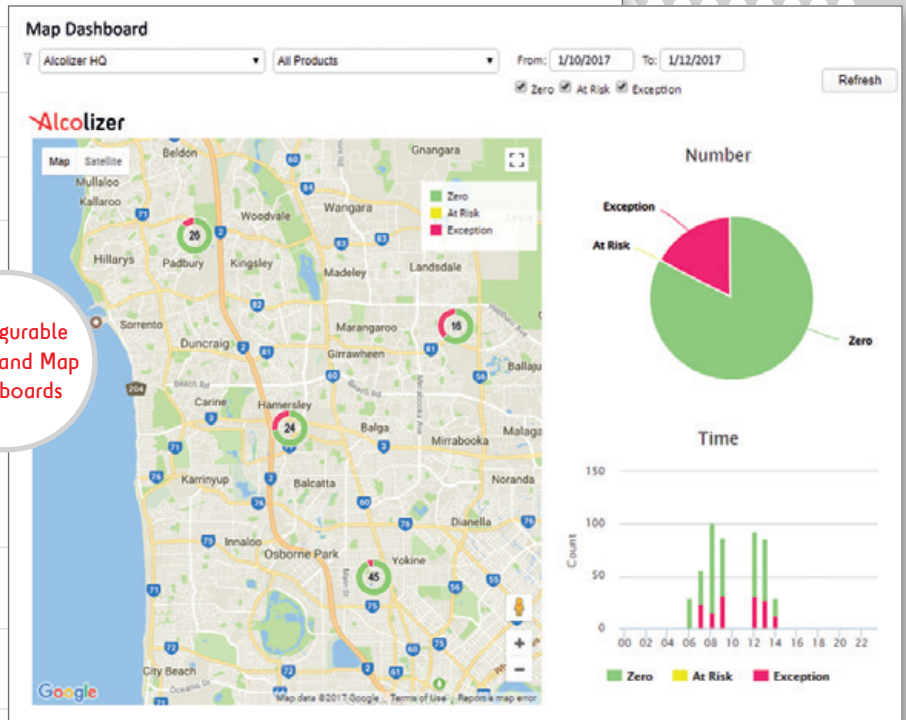

Above-Left: AlcoCONNECT™ Live Dashboards (Graph and Map) with Alcolizer and Druglizer summaries.

Activities

All sites All Products From: 25/10/2017 To: 25/12/2018 Export

Staff ID	Staff Name	Site	Product	Time	BAC	Serial	Location
1033	Joe Bloggs	Balcatta	LE5	17:04:57	0.000 g/100mL BAC	12345678	Warehouse office
2450	Ally Kay Lizer	Padbury	LE5	14:15:03	0.000 g/100mL BAC	90123456	Padbury depot
1248	Jan Bloggs	Balcatta	LE5	14:19:31	0.000 g/100mL BAC	78901234	Warehouse office
6250	Karl Wright	Malaga	LE5	2017-10-25			
3267	Adrian White	Osbourne Pk	LE5	2017-10-25			
1102	Jane Doe	Malaga	LE5	2017-10-25			
5986	Tess Toeur	Osbourne Pk	LE5	2017-10-25			
9121	Jill Bloggs	Padbury	LE5	2017-10-25			
8574	Joe Bloggs	Balcatta	LE5	2017-10-25			
3316	Jim Bloggs	Padbury	LE5	2017-10-25			
2518	Jan Bloggs	Balcatta	LE5	2017-10-25			
4596	Jeff Bloggs	Malaga	LE5	2017-10-25			
3318	Joe Bloggs	Osbourne Pk	LE5	2017-10-25			
2703	Jack Bloggs	Malaga	LE5	2017-10-25			

Drill-down into Activities List

Activity Record

The breath test result as it was displayed

Result: 0.000 g/100mL BAC
Date and Time: 25-10-2017 14:19:31
Log Record ID: 9439

Machine Identification

Serial Number: 90123456
Product: WM4 Balcatta
Description: Balcatta
Assembly Code: AS-00003-04
Software Version: ac-12.1

Staff

Staff ID: 1248
Name: Jan Bloggs
Job Title: Driver
Phone: 1300 789 908

Time stamped Activity Record



Above: Activities List showing test summaries. Right: An Activity Record with camera capture.

MARKET LEADING SOLUTIONS FOR YOUR BUSINESS

Wall Mount 4 alcohol tester

Delivering superior alcohol breath testing solutions for commercial and heavy use industrial customers around the world. Enable with the AlcoCONNECT™ Live Data Management system.

- ✓ Fast testing of high workforce volumes
- ✓ Efficient, reliable, durable operation
- ✓ Configurable to your Drug and Alcohol program
- ✓ Instant results to management
- ✓ Self calibrating every 24 hours
- ✓ Real peace of mind for your business



Integration options include:



OnSite Testing alcohol & drug solutions

The OnSite Testing Alcohol and Drug Solutions App is the ideal tool for Safety and Business Managers seeking real-time test results. Enable with AlcoCONNECT™ Live Data Management system for the total solution.

- ✓ Test where and when you need
- ✓ Easy to input donor information and acknowledgment
- ✓ Auto Bluetooth connectivity of results to App
- ✓ Email results or sync with AlcoCONNECT™ Data Management
- ✓ Option available for urine test result collection

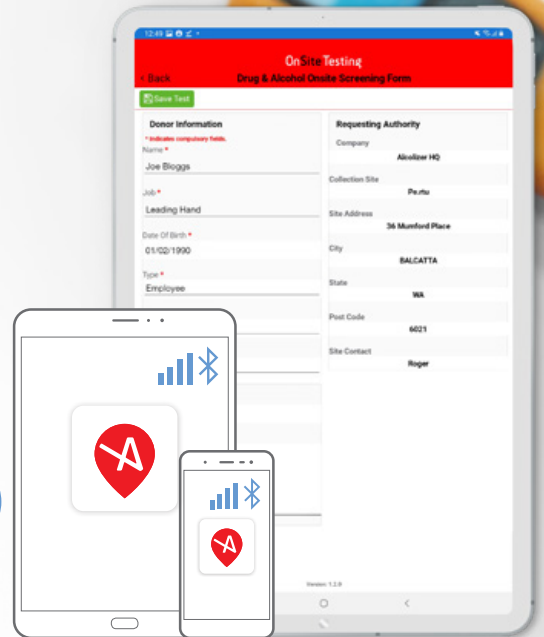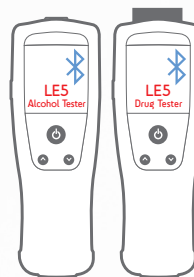

Contact any of our nationwide offices by calling **1300 789 908** or email sales@alcolizer.com

Alcolizer
technology

TRUSTED ALWAYS