

# OnSite Testing App (Android) Bluetooth Quick User Guide



Alcolizer LE5  
Breathalyser

OnSite Testing  
alcohol & drug solutions


Alcolizer  
technology  
TRUSTED ALWAYS

## Pairing Instructions

### Step 1


Access the tablet/phone Settings > Connections > Bluetooth (BT) menu. Ensure BT is on.

Remain on this page while performing step two.

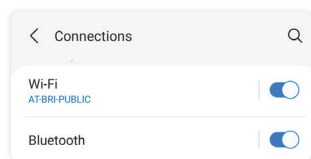

### Step 2


Access the LE5 Breathalyser Setup Menu: press the Λ & V buttons together on the idle screen for 4 seconds until a code entry screen appears. Enter code 00005000.

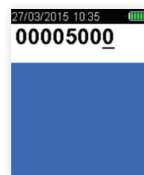

### Step 3

Navigate to the Bluetooth Comms menu > Enable BT if not already. Scroll down to and select 'Serial' option.

Ensure the 'Auto Send Standard' and 'Auto Send Passive' options are set to 'All' if that option is required.

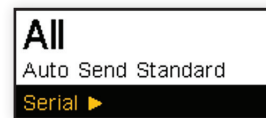

### Step 4

Scroll up to and select the 'Pairing' option. Devices will start to populate the screen.

Select the tablet/phone on the LE5 device.



### Step 5

The tablet/phone will display a prompt with a password. Select 'Pair' on the tablet.


The LE5 device will show 'Paired' on its screen.



### Step 6

Open the Onsite App. For 'Quick Test Mode', begin testing.

For 'Full Test Mode', go to 'Settings', select the LE5 device, set expiry date and begin testing.

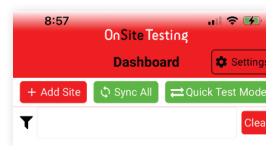

## Troubleshooting Quick Guide


Result Failed to Send? Possible cause for error:

- ▶ Paired to incorrect tablet/phone
- ▶ Devices are not paired
- ▶ Lost connection - requires repairing
- ▶ Stale Bond

Follow the below steps to re-establish a fresh connection between devices.


### Step 1

On the tablet/phone, select Settings > Connections > Bluetooth. Select the icon next to the LE5 serial number and 'unpair' on the bottom left of page.

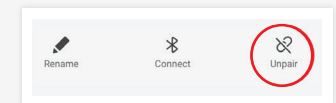

### Step 2

On the LE5, follow step 2 of the **Pairing Instructions** (see left hand side), enter the 'Setup' menu. Select BT Comms menu > Serial > Forget This Device. The message 'No device paired' will display.

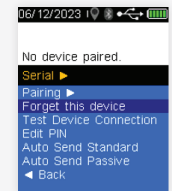

### Step 3

Follow steps 1 – 6 of the **Pairing Instructions** (see left hand side) to re-establish a connection between devices.

Scan this code for Video assistance.



# OnSite Testing App (Android) Testing Session Quick Guide


**Alcolizer LE5  
Breathalyser**



## Testing Instructions


### Step 1

Select the appropriate 'Site' from the dashboard, listed in white boxes with the Site name.

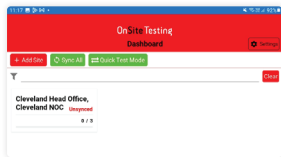


### Step 2

Select 'Add Test' on the top left. Complete all donor information. All fields are required.

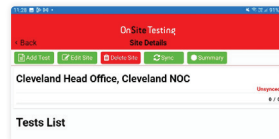

### Step 3

Tap the Donor Signature section to select test type and donor medication if any. Donor must sign the Agreement.

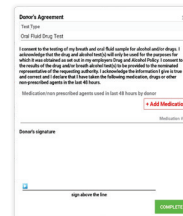


### Step 4

Select Alcohol Test once donor information has been completed. Complete the alcohol screening using the LE5 Breathalyser.




### Step 5

The result will transfer to the app and populate the results screen.

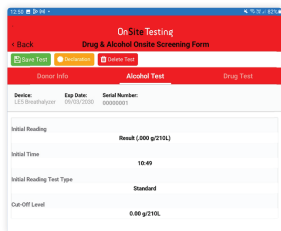


### Step 6

The donor must sign the Declaration form.

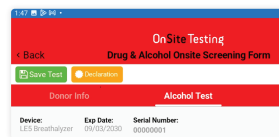


The Certification form will then pop up for the Collector to sign.

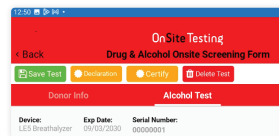

### Step 7

The app will now display the Site section again and the test listed as completed in green. Follow the above steps if completing more than one test.

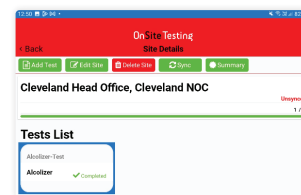


### Step 8

Once the testing session is complete, sign the Summary form. Both signatures are required. After signing, select 'back' on the top left of the form.



### Step 9


Select 'Sync' on the top of the screen. Ensure you have internet connection. Access the AlcoCONNECT platform to view results and reports.

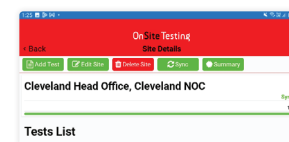

## Quick Test Mode


### Step 1

Select 'Quick Test Mode' from the Dashboard screen.

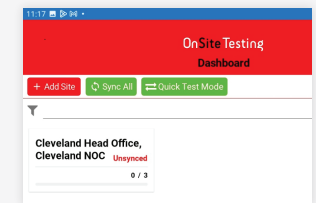

### Step 2

Enter the staff ID if required. Conduct an alcohol screening test.

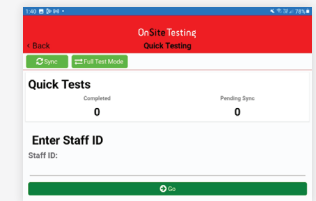

### Step 3

Access the AlcoCONNECT platform to view results and reports.

