

# OnSite Testing App (Android) Bluetooth Quick User Guide



Alcolizer LE5  
Druglizer

OnSite Testing  
alcohol & drug solutions


Alcolizer  
technology  
TRUSTED ALWAYS

## Pairing Instructions

### Step 1


Access the tablet/phone Settings > Connections > Bluetooth (BT) menu. Ensure BT is on.

Remain on this page while performing step two.

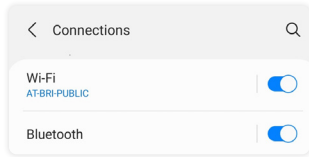


### Step 2

Access the Druglizer Main Menu: press the Λ & V buttons together on the idle screen and navigate to Configuration > AlcoCONNECT > Configure

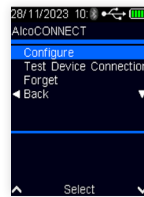

### Step 3

Press the V button, ensure BT is enabled if not already. Press the V button again until page 5 is displayed. The screen will start to populate with devices.

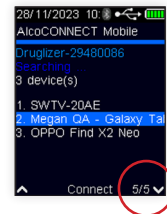

### Step 4

Select the tablet/phone on the Druglizer device.



### Step 5

The tablet/phone will display a prompt with a password. Select 'Pair' on the tablet.


The Druglizer device will show 'Paired' on its screen.



### Step 6

For 'Full Test Mode', go to 'Settings', select the Druglizer device, set expiry date and begin testing.

*Quick Test Mode is currently not supported in Druglizer.*

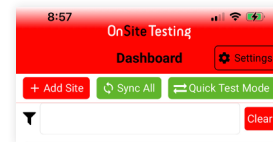


## Troubleshooting Quick Guide


Result Failed to Send? Possible cause for error:

- ▶ Paired to incorrect tablet/phone
- ▶ Devices are not paired
- ▶ Lost connection - requires repairing
- ▶ Stale Bond

Follow the below steps to re-establish a fresh connection between devices.


### Step 1

On the tablet/phone, select Settings > Connections > Bluetooth. Select the  icon next to the Druglizer serial number and 'unpair' on the bottom left of page.

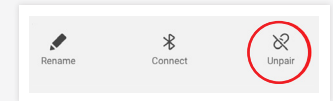

### Step 2

Access the Druglizer Main Menu: press the Λ & V buttons together on the idle screen and navigate to Configuration > AlcoCONNECT > Forget. Select Forget. The Druglizer and Android phone/tablet are now unpaired.

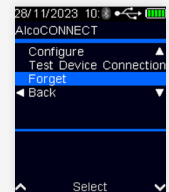

### Step 3

Follow steps 1 – 6 of the **Pairing Instructions** (see left hand side) to re-establish a connection between devices.



# OnSite Testing App (Android) Testing Session Quick Guide


**Alcolizer LE5  
Druglizer**

**Alcolizer**  
technology  
TRUSTED ALWAYS

## Testing Instructions


### Step 1

Select the appropriate 'Site' from the dashboard, listed in white boxes with the Site name.

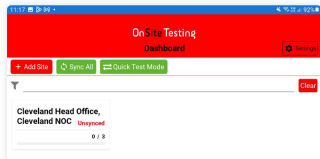

### Step 2

Select 'Add Test' on the top left. Complete all donor information. All fields are required.

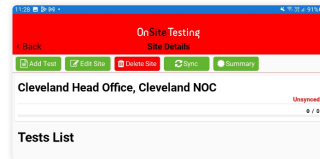

### Step 3

Tap the Donor Signature section to select test type and donor medication if any. Donor must sign the Agreement.



### Step 4

Select Drug Test once donor information has been completed.

Complete the drug screening using the LE5 Druglizer.




### Step 5

The result will transfer to the app and populate the results screen.




### Step 6

The donor must sign the Declaration form.

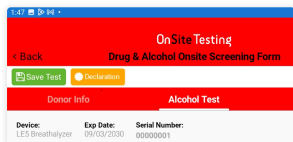


The Certification form will then pop up for the Collector to sign.

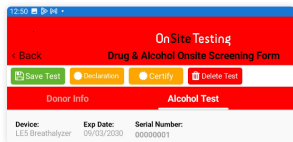

### Step 7

The app will now display the Site section again and the test listed as completed in green.

Follow the above steps if completing more than one test.

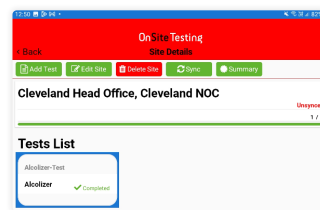

### Step 8

Once the testing session is complete, sign the Summary form. Both signatures are required. After signing, select 'back' on the top left of the form.

### Step 9

Select 'Sync' on the top of the screen. Ensure you have internet connection.

Access the AlcoCONNECT platform to view results and reports.

