

OnSite Testing App (iOS) Bluetooth Quick User Guide



OnSite Testing
alcohol & drug solutions

Alcolizer
technology
TRUSTED ALWAYS

Pairing Instructions


Step 1

Access the iPad/iPhone Settings > Bluetooth menu. Ensure BT is on.




Step 2

Access the Druglizer Main Menu: press the Λ & V buttons together on the idle screen and navigate to Configuration > AlcoCONNECT > Configure

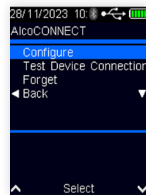

Step 3

Press the V button, ensure BT is enabled if not already. Press the V button again until page 5 is displayed. The screen will start to populate with devices.

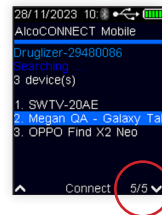

Step 4

DO NOT select a device from this screen. It will be done from the App.



Step 5

Open the App, navigate to Settings > Pairing > Scan


The App will search for the Druglizer serial number. When it is displayed, select "Pair". The iPad/iPhone will request the pairing pin. Enter the pin displayed on the Druglizer. Select Pair. The Druglizer will display "Paired".



Step 6

For 'Full Test Mode', go to 'Settings', select the Druglizer device, set expiry date and begin testing.

Quick Test Mode is currently not supported in Druglizer.

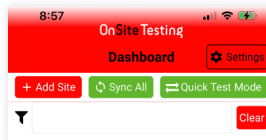

Troubleshooting Quick Guide


Result Failed to Send? Possible cause for error:

- ▶ Paired to incorrect tablet/phone
- ▶ Devices are not paired
- ▶ Lost connection - requires repairing
- ▶ Error Message (e.g. stale bonding)

Follow the below steps to re-establish a fresh connection between devices.


Step 1

On the iPad/iPhone, select Settings > Bluetooth. Select the ⓘ icon next to the Druglizer serial number and select 'Forget This Device'.

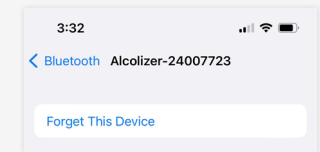

Step 2

Access the Druglizer Main Menu: press the Λ & V buttons together on the idle screen and navigate to Configuration > AlcoCONNECT > Forget. Select Forget. The Druglizer and Android phone/tablet are now unpaired.

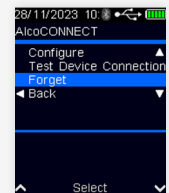

Step 3

Follow steps 1 – 6 of the **Pairing Instructions** (see left hand side) to re-establish a connection between devices.



OnSite Testing App (iOS) Testing Session Quick Guide


**Alcolizer LE5
Druglizer**

Alcolizer
technology
TRUSTED ALWAYS

Testing Instructions


Step 1

Select the appropriate 'Site' from the dashboard, listed in white boxes with the Site name.

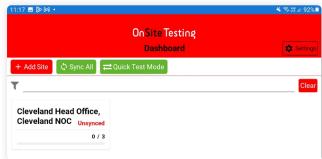

Step 2

Select 'Add Test' on the top left. Complete all donor information. All fields are required.

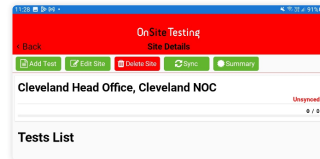

Step 3

Tap the Donor Signature section to select test type and donor medication if any. Donor must sign the Agreement.



Step 4


Select Drug Test once donor information has been completed.

Complete the drug screening using the LE5 Druglizer.




Step 5

The result will transfer to the app and populate the results screen.

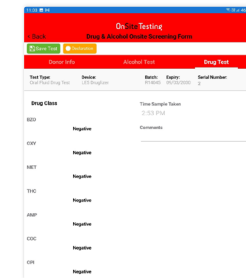


Step 6

The donor must sign the Declaration form.

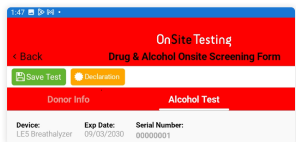


The Certification form will then pop up for the Collector to sign.

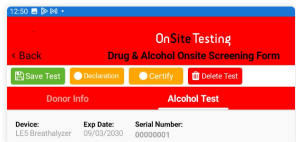

Step 7

The app will now display the Site section again and the test listed as completed in green.

Follow the above steps if completing more than one test.

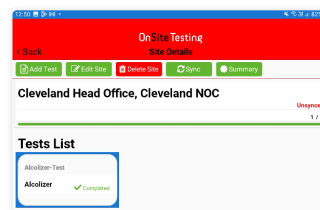

Step 8

Once the testing session is complete, sign the Summary form. Both signatures are required. After signing, select 'back' on the top left of the form.

Step 9

Select 'Sync' on the top of the screen. Ensure you have internet connection.

Access the AlcoCONNECT platform to view results and reports.

