

OnSite Testing App (iOS) Bluetooth Quick User Guide



Alcolizer LE5
Breathalyser

Alcolizer
technology
TRUSTED ALWAYS

Pairing Instructions


Step 1

Access the iPad/iPhone Settings > Bluetooth menu. Ensure BT is on.



Step 2


Access the LE5 Breathalyser Setup Menu: press the Λ & V buttons together on the idle screen for 4 seconds until a code entry screen appears. Enter code 00005000.

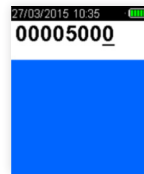

Step 3

Navigate to the Bluetooth Comms menu > Enable BT if not already. Scroll down to and select 'Serial' option.

Ensure the 'Auto Send Standard' and 'Auto Send Passive' options are set to 'All' if that option is required.

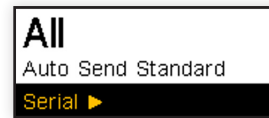

Step 4

Scroll up to and select the 'Pairing' option. Devices will start to populate the screen.

DO NOT select a device from this screen. It will be done from the App.



Step 5

Open the App, navigate to Settings > Pairing > Scan

The App will search for the LE5 serial number. When it is displayed, select "Pair". The iPad/iPhone will request the pairing pin. Enter the pin displayed on the LE5. Select Pair. The LE5 will display "Paired".




Step 6

In Settings > select Instruments and update the Expiry and Calibration Dates.

Step 7

Navigate to the App Dashboard by selecting the 'Back' option. Select 'Quick Test Mode' and begin testing.

For 'Full Test Mode', in 'Settings', select the LE5 device, set expiry date and begin testing.

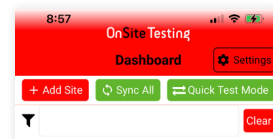

Troubleshooting Quick Guide


Result Failed to Send? Possible cause for error:

- ▶ Paired to incorrect tablet/phone
- ▶ Devices are not paired
- ▶ Lost connection - Requires repairing
- ▶ Error Message (e.g. stale bonding)

Follow the below steps to re-establish a fresh connection between devices.


Step 1

On the iPad/iPhone, select Settings > Bluetooth. Select the ⓘ icon next to the LE5 serial number and select 'Forget This Device'.

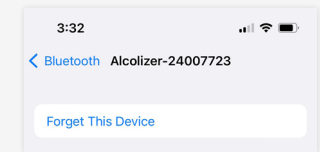

Step 2

On the LE5, follow step 2 above to enter the 'Setup' menu. Select BT Comms menu > Serial > Forget This Device. The message "No device paired" will display.

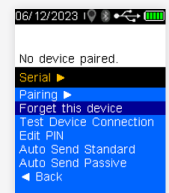

Step 3

Follow steps 1 – 6 of the **Pairing Instructions** (see left hand side) to re-establish a connection between devices.

Scan this code for Video assistance.



OnSite Testing App (iOS) Testing Session Quick Guide


**Alcolizer LE5
Breathalyser**

Alcolizer
technology
TRUSTED ALWAYS

Testing Instructions


Step 1

Select the appropriate 'Site' from the dashboard, listed in white boxes with the Site name.

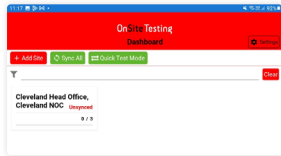

Step 2

Select 'Add Test' on the top left. Complete all donor information. All fields are required.

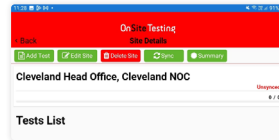

Step 3

Tap the Donor Signature section to select test type and donor medication if any. Donor must sign the Agreement.




Step 4

Select Alcohol Test once donor information has been completed. Complete the alcohol screening using the LE5 Breathalyser.




Step 5

The result will transfer to the app and populate the results screen.

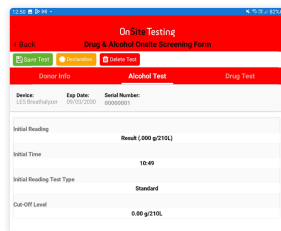


Step 6

The donor must sign the Declaration form.

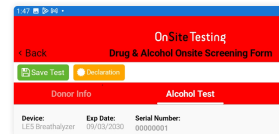

The Certification form will then pop up for the Collector to sign.

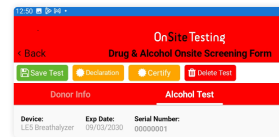


Step 7

In Settings > select Instruments and update the Expiry and Calibration Dates.

Step 8

The app will now display the Site section again and the test listed as completed in green. Follow the above steps if completing more than one test.

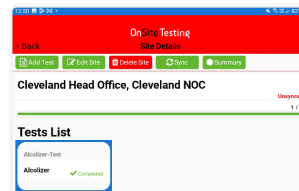


Step 9

Once the testing session is complete, sign the Summary form. Both signatures are required. After signing, select 'back' on the top left of the form.

Step 10


Select 'Sync' on the top of the screen. Ensure you have internet connection. Access the AlcoCONNECT platform to view results and reports.

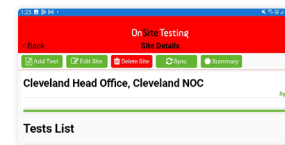

Quick Test Mode


Step 1

Select 'Quick Test Mode' from the Dashboard screen.

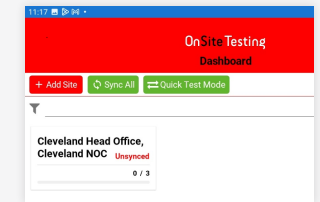

Step 2

Enter the staff ID if required. Conduct an alcohol screening test.

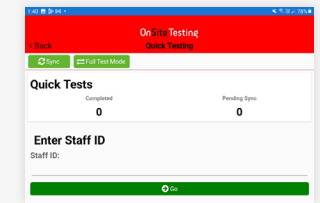

Step 3

Access the AlcoCONNECT platform to view results and reports.

